Stille Glück

NUK e-Motion Elektrische Milchpumpe

NUK e-Motion Electric Breast Pump

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Käufer:

Contents

1. Important Information Before Using for the First Time	16
2. Description	16
2.1 Intended purpose	16
2.2 Function	16
2.3 Contraindications	16
2.4 Side effects	16
2.5 Symbols on the pump	17
3. Safety Information	17
3.1 General information	17
3.2 Hygiene	18
3.3 NUK Breast Milk Container	18
3.4 NUK Vented Teat	18
4. Names of Parts (Figure ④)	19
5. Cleaning and Sterilising	19
5.1 Cleaning intervals	19
5.2 Cleaning (Figure ① in reverse order)	19
5.3 Sterilising	20
6. Assembly (Figure ①)	20
7. Power Supply (Figure ②)	20
7.1 Mains operation	20
7.2 Interchangeable plug attachments	20
7.3 Battery operation	21
8. Operation (Figure ③)	21
8.1 Feeding expressed milk	22
8.2 Storing expressed milk	22
9. Tips for Successful Expressing	22
9.1 Frequency of expressing	22
9.2 Procedure	22
9.3 Milk quantity	23
9.4 Milk flow reflex	23
10. Hygiene Tips	23
11. Storing Milk	24
11.1 NUK Breast Milk Container	24
11.2 Tips for handling breast milk	24
12. Troubleshooting	25
13. Technical Data	25
14. Disposal	25
15. Warranty	26
16. Warranty Certificate	26

Dear Mother,

For babies, a mother's milk is the most important lifesource there is. That is why NUK developed a breastfeeding programme to provide practical and convenient breastfeeding aids for your every need so that you and your baby can enjoy breastfeeding. This easy-to-handle breast pump is designed to help you and your baby whenever natural breast-feeding is not possible.

Please take 5 minutes of your time to carefully read these operating instructions before using your NUK Electric Breast Pump for the first time.

We wish you and your baby every happiness.

I. Important Information Before Using for the First Time

Clean and sterilise all **accessories** (except the motor unit **12** and mains adapter **18**) before using the

NUK Electric Breast Pump for the first time (see "5. Cleaning and Sterilising" on page 19).

2. Description

2.1 Intended purpose

The NUK Electric Breast Pump is an aid to expressing and collecting breast milk:

- in the workplace or when baby is away, at home or when travelling
- to maintain milk production during a break in breast feeding
- to relieve engorgement.

Once expressed, breast milk can be collected, stored, frozen, warmed and fed to your baby in the NUK Breast Milk Container.

2.2 Function

The NUK Electric Breast Pump has an electric suction pump and offers the following benefits:

- a 2-stage pump system imitates your baby's natural sucking action
- easily adjustable suction strength
- soft, close-fitting breast shield
- for your ease both mains and battery operation (mains adapter supplied).

2.3 Contraindications

If the breast is inflamed, infected or sore the NUK Electric Breast Pump should preferably not be used or only used with special care. If you are unsure at any time whether to use the NUK Electric Breast Pump, please consult your health visitor or doctor.

2.4 Side effects

If at any time you experience an unexpected reaction, please consult your midwife, health visitor, doctor or pharmacist.

2.5 Symbols on the pump

In the motor unit (battery compartment)	
Symbol	Meaning
	• When inserting batteries, check for correct pole alignment
	• Do not dispose of batteries or the pump with household waste
	• Refer to the operating instructions
	• Refer to accompanying documents
	• Serial number

On the motor unit	
BP4	• Model name
	• Complies with all the requirements of applicable EC directives
	• Electrically protected device, Protection Class II
	• Electromedical device, Type BF
	• Refer to accompanying documents
	• Direct current 4.5 V, max. 0.5 A

3. Safety Information

For your own safety please note the following points:

3.1 General information

- Please carefully read the operating instructions before using the pump. They are part of the device and must be kept available at all times.
- Use the NUK Electric Breast Pump only for the purpose described (see "2.1 Intended purpose" on page 16).
- Electric medical products such as this must only be used by adults.
- The breast pump must be operated using only the adapter supplied.
- Check that the mains voltage required for the adapter **18** (see type plate) is compatible with your mains voltage.
- Never immerse the motor unit **12** in water as there is a danger of electrocution and the motor unit may become damaged if water leaks into the unit.
- Do not expose the device to direct sunlight as UV rays and overheating may make the plastic brittle.
- Keep small parts that may be swallowed out of the reach of children as there may be a danger of choking.
- If ointments or creams are used, please check in the manufacturer's information whether the ointment or cream needs to be removed before nursing or expressing milk.
- The batteries must be removed when flat or if the device is not going to be used for a while as leaking batteries may damage the device.
- Do not use rechargeable batteries as their voltage is too low for the pump to work to its full capacity.
- Please note that any claim under the warranty or for liability will be invalid if accessories are used that are not recommended in these operating instructions, or if original spare parts are not used for repairs. This also applies to repairs undertaken by non-qualified persons.
- If accessories from other manufacturers are used, the pump may not function properly.
- If the mains connection lead for this device is damaged, it must not be repaired. Instead,

the entire mains adapter must be replaced in order to avoid potential danger.

- In the event of any malfunction, the pump must be returned to the manufacturer for repair. Failure to do so will invalidate your warranty.
- This device meets EU EMC standards. Nonetheless there is the possibility of interaction with other electrical devices. You should therefore keep it away from other devices during operation.
- Never open the motor unit housing **12 as this will invalidate the warranty.**
- If the breast milk is intended for a premature baby, it is essential that you follow your doctor's advice.

3.2 Hygiene

- When handling breast milk, hygiene is of particular importance to ensure your baby's healthy development.
- Please refer to section "5. Cleaning and Sterilising" on page 19 to prevent infection or bacterial contamination and to prevent the device from becoming damaged.
- **The breast pump must be cleaned thoroughly to ensure its full suction strength.**
- Never open the motor unit housing **12 as this will invalidate the warranty.**

3.3 NUK Breast Milk Container

- If the bottle is damaged on the inside (e.g. by scratches), it should be replaced for hygiene reasons.
- All unused individual parts should always be kept out of the reach of children as there may be a danger of choking.
- Tea or juices may discolour bottles and accessories.
- Give only the quantities of food and drink recommended by the manufacturer and in the recommended consistency.

- Sterilisation in the microwave is not recommended as this can damage the materials.
- Feeding bottles that are sealed and contain food should not be put in the microwave as there may be a risk of explosion or material damage. Uneven heating may also result in scalding.
- NUK bottles and teats are only compatible with the relevant system **NUK FIRST CHOICE.**

3.4 NUK Vented Teat

- Clean and sterilise the teat thoroughly before using for the first time and before each further use.
- The NUK Steam Steriliser is recommended for sterilising.
- When boiling, check that there is sufficient liquid in the steriliser in order to avoid damage (5 minutes boiling time is sufficient).
- The teat must not be cleaned in a dishwasher or sterilised in a microwave as it could be damaged by the dishwasher detergent or deformed.

For your child's health and safety: **CAUTION!**

- Store in a hygienic condition and away from light sources.
- Always check food temperature before feeding.
- This product must only be used under adult supervision. A teat should never be used as a soother. Continuous and prolonged sucking of fluids may cause tooth decay.
- Do not dip the teat in sweetened liquids or medicine.
- **Check the teat before each use by pulling it in all directions – especially when children are teething. Discard the teat at the first sign of tearing or biting damage as there is a danger that small parts may become detached and swallowed.**

- Do not leave teats in cold sterilising solution beyond the period specified by the manufacturer, as this may cause damage to the material.
- For hygiene and safety reasons, teats should be replaced every 1-2 months.

- Use teats only with NUK FIRST CHOICE bottles. The NUK Vented Teat and NUK Breast Milk Container comply with EN 14350.

4. Names of Parts (Figure ④)

- 1 Valve
- 2 Breast shield
- 3 Connection ring
- 4 Lip valve
- 5 Breast milk container
- 6 Stand for breast milk container
- 7 Sealing disc
- 8 Teat
- 9 Screw ring
- 10 Protective cap

- 11 Power supply plug
- 12 Motor unit
- 13 Warning light when the voltage is too low
(batteries need changing)
- 14 On/off button
- 15 Battery compartment catch lock
- 16 Battery compartment lid
- 17 Suction power regulator
- 18 Mains adapter
- 19 Interchangeable plug attachments

5. Cleaning and Sterilising

Hygiene is of priority when handling breast milk and all parts that come into contact with it. The pump should therefore only be used in a clean and sterile condition.

5.1 Cleaning intervals

After each use:

- All accessories except for the motor unit 12 and adapter 18 must be thoroughly cleaned with water and sterilised immediately after each use.

5.2 Cleaning (Figure ① in reverse order)

Caution! Unplug the mains adapter 18 from the wall socket before cleaning.

Caution! Clean the lip valve 4 carefully. Hard and pointed bristles may damage the valve.

1. Unscrew the motor unit 12 together with the connection ring 3 from the bottle 5.

2. Remove the lip valve 4 from the lower part of the breast shield.
3. Unscrew the connection ring 3 from the motor unit 12.
4. Pull the motor unit 12 off the breast shield 2.
5. Remove the valve 1 from the breast shield 2.
6. All accessories apart from the motor unit 12 and adapter 18 should be cleaned and rinsed under running water as milk residues may result in the build up of bacteria and may damage the material.

Note: All accessories, apart from the motor unit 12, teat 8 and adapter 18 may be cleaned in a dishwasher.

7. Wipe the motor unit 12 with a damp cloth.
Caution: Never immerse the motor unit or adapter in water as this could damage the electrics.

5.3 Sterilising

Before sterilising, clean as described in section "5.2 Cleaning (Figure ① in reverse order)".

Important! The motor unit 12 and adapter 18 should never be sterilised as this may damage the electrics.

- Use a NUK Steam Steriliser, observing the relevant operating instructions, or sterilise

the parts in boiling water for 5 minutes. Ensure there is sufficient water in the saucepan to avoid damage.

Caution! Parts should not be sterilised in a microwave, as this can result in damage to the material.

Important! Ensure that the accessories and parts have completely cooled before using again.

6. Assembly (Figure ①)

1. Insert the valve 1 into the breast shield 2 with the knob pointing upwards.
2. Position the motor unit 12 on the breast shield 2.
3. Screw the connection ring 3 from below onto the motor unit 12.
4. Insert the lip valve 4 from below into the assembled breast shield 2.
5. Screw together the assembled motor unit 12 and the bottle 5.

7. Power Supply (Figure ②)

7.1 Mains operation

For **mains operation** use only the enclosed adapter, model KK 07-045-0800.

1. Open the battery compartment by the catch lock 15.

2. Insert the plug 11 on the adapter 18 into the socket in the battery compartment.

3. Close the battery compartment again.

Push both the "feet" on the lid into the relevant recess on the motor unit and shut the lid, ensuring the catch lock 15 engages with an audible click.

Note: The device should not be operated without the battery compartment lid.

4. Fit the interchangeable plug attachment 19 suitable for your power supply to the adapter.

You can find out which plug is suitable for your country from the following section.

5. Plug the mains plug 18 into a wall socket.

7.2 Interchangeable plug attachments


The mains adapter is supplied with an EU type plug. Additional interchangeable plugs are also included. You can use these to operate your breast pump when abroad as follows:

	Commonwealth, Africa, Asia, Great Britain, Middle East etc.
	Australia, New Zealand, China
	Europe, South America, Africa etc.
	North, Central and South America, Japan, South East Asia etc.


Changing the plug

1. Using an item such as a ball-point pen, press on the release catch on the plug.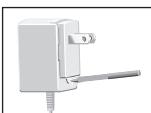

2. Slide the plug about 1.5 cm upwards and then remove it to the front.

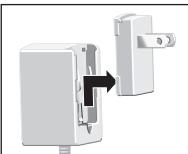3. Position the interchangeable plug attachment so that the grooves on the attachment meet the contacts on the adapter.

4. Slide the plug attachment downwards until the locking mechanism clicks audibly into place.

7.3 Battery operation

Battery operation requires 3 Type AAA, 1.5V batteries.

Rechargeable batteries should not be used to operate the pump. Rechargeable batteries only have a voltage of 1.2V; disposable batteries have 1.5V. The lower voltage of rechargeable

batteries means that the pump underperforms. Moreover, any rechargeable batteries that may be inserted will not be recharged.

The warning light **I3** will warn you that the voltage is too low when the pump is operated on battery power. As soon as this light shines red you need to change the batteries. When the batteries are fully discharged the warning light **I3** can no longer light up.

1. Open the battery compartment by the catch lock **I5**.
2. Insert the batteries, ensuring that they are positioned correctly according to the + and - symbols in the battery compartment!
3. Close the battery compartment again.

Push both the "feet" on the lid into the relevant recess on the motor unit and shut the lid, ensuring the catch lock **I5** engages with an audible click.


Note: The device should not be operated without the battery compartment lid.

Caution! The batteries should be removed if the device is not going to be used for a while to prevent the possibility of leaking batteries causing damage to the device.

8. Operation (Figure ③)

Caution! As hygiene is of upmost importance, please always wash your hands before expressing milk and observe the advice in "10. Hygiene Tips" on page 23.

Important!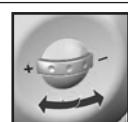

- = low suction

+ = high suction

First set the suction power regulator **I7** to minimum by rotating the ring towards the "-" symbol. You can then easily increase the suction strength as you express milk in Phase 2.

1. Position the breast shield **2** on your breast so that the nipple is positioned in the centre


of the shield. Gently press the breast shield in order to ensure a good seal when expressing the milk. **Hold the pump in a vertical position.**

Note! If you hold the pump at too great an angle or if the pump topples over, it will switch off automatically.

2. Switch the breast pump on. To do this, press the On/Off button **14**.

Important! The breast pump automatically simulates your baby's natural sucking action: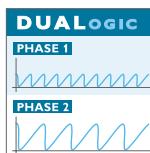

Phase 1: In the first minute the sucking rhythm is short and stimulating to stimulate the milk flow reflex.

Phase 2: After this the pump changes to a slow and intensive sucking rhythm.

Important! If the pump is switched off and then switched on again the programme goes back to the beginning with the fast suction rhythm. This gives you the opportunity to extend Phase 1.

3. Now you can adjust the suction using the suction power regulator **17** until the optimum suction strength is reached.

If the suction is too weak, rotate towards the "+" symbol.

If the suction is too strong, rotate the suction power regulator **17** towards the "—" symbol.

Note: In phase 2 the breast pump automatically changes to a stronger suction power, so remember to select a lower suction setting at the beginning.

Caution: In order not to damage the pump you should ensure that the expressed milk does not reach as far as the lip valve **4**.

4. Press the On/Off button **14** again to stop the pumping action.
5. After use, place the breast pump in the stand **6** for stability.
6. Unscrew the bottle **5** from the connection ring **3** and motor unit. Be sure to keep the bottle vertical.

8.1 Feeding expressed milk

1. Screw the teat **8** onto the bottle with the screw ring **9**.

8.2 Storing expressed milk

1. To store the milk, secure the bottle using the supplied screw ring **9** and sealing disc **7**.

Caution: Please observe the information in section "11. Storing Milk" on page 24.

2. Using a freezer pen, write the date and time of storage on the label on the container.

9. Tips for Successful Expressing

Correct expressing needs practice. You should first become accustomed to your new pump.

9.1 Frequency of expressing

The frequency of expressing depends on your individual requirements.

If you are apart from your baby and/or want to sustain milk production, the natural breastfeeding rhythm should be maintained. You should therefore express 6 to 8 times daily, including once during the night.

In all other cases you should express entirely to suit your own individual needs.

9.2 Procedure

Express milk from each breast in turn.

This electric breast pump's DUALogic 2-phase pump system imitates your baby's natural sucking action.

Milk is expressed at short intervals for about one minute. This simulates your baby's sucking action which is rapid at first in order to quench the baby's thirst. At the same time, rapid sucking stimulates the milk flow.

After this the breast pump changes automatically to a slow, more intensive pumping rhythm. This corresponds to your baby's drinking behaviour. As soon as baby's thirst is quenched it changes to a calm, slow and intensive sucking action so as to drink its fill.

Alternate between breasts when the milk flow slows down or ceases.

The amount of time needed to express milk will vary considerably from one person to another.

9.3 Milk quantity

Please do not be surprised if initially it takes a while for the milk to start to flow. This is not unusual. In fact it is quite normal for the following reasons:

Milk production will adapt itself to daily requirements.

This means that the quantity of milk produced may vary considerably.

- Always ensure that you drink plenty of fluids. This helps to stimulate milk production.
- Do not set the suction strength too high. A higher suction strength does not increase the quantity of milk produced.

9.4 Milk flow reflex

With breastfeeding, putting your baby to the breast stimulates the milk flow reflex and the milk will begin to flow. You can also stimulate the milk flow reflex as follows:

- Find yourself a quiet place to sit, where you can relax fully when using the breast pump.
- Find a comfortable and relaxed sitting position.
- Before starting, place a warm object (e.g. NUK Gel Breast Cushion) on your breasts for a few minutes to help relax the tissue.
- Make sure you have eye contact with your baby. If that is not possible, look at a photo of your baby.
- The milk flow reflex releases the flow of milk to both breasts simultaneously. Any surplus milk leaking from the breast that is not currently being expressed can be collected using the NUK Milk Capturing Dish.

10. Hygiene Tips

Hygiene is particularly important when using the breast pump. This applies to all the individual pump components as well as to your own personal hygiene.

We recommend a daily shower or washing your nipples in clean water. Where possible, avoid using soap on your nipples, as this may cause irritation.

II. Storing Milk

If the breast milk is not to be used immediately, you should seal the filled bottle well and store it in the refrigerator.

Caution! Only the sealed breast milk container should be stored in the refrigerator, not the entire NUK Electric Breast Pump.

II.1 NUK Breast Milk Container

The breast milk may be chilled, frozen and thawed out again in the breast milk container. The container may also be used for re-heating and feeding. Please always observe the information below on handling breast milk.

Note: When freezing breast milk, do not fill the container with more than 150 ml (5 fl.oz), as breast milk expands as it freezes and the container may burst.

II.2 Tips for handling breast milk

Breast milk	Storage location	Storage periods
Freshly expressed	At room temperature	Max. 6-8 hours
	Refrigerator (not in door)	72 hours at 4 °C
	Refrigerator freezer compartment	2 weeks
	Upright/chest freezer	6 months at -18 °C
Defrosted, unopened	Refrigerator (not in door)	24 hours at 4 °C
Defrosted, opened	Refrigerator (not in door)	12 hours at 4 °C
Defrosting milk	Gradually in refrigerator	Approx. 24 hours at 4 °C
	At room temperature followed by immediate feeding	
	If time is short, under running cold water or lukewarm water (max. 37 °C), then feed	

- To attain the correct feed temperature of approx. 37 °C, place the container in a NUK Baby Food Warmer.
- Note:** The container must be open to prevent pressure building up as the milk is warmed.
- Caution:** Breast milk must not be thawed out in a microwave as this will destroy valuable nutrients as well as causing possible damage to the container material, which may cause the risk of explosion. Uneven heating may also result in scalding to your baby.
- The milk should therefore be shaken gently before feeding to ensure that milk and milk fats are well mixed and that the warm breast milk is evenly distributed.

- Defrosted, re-heated or unused milk must be disposed of and not refrozen.
- If you express breast milk more than once within a period of 24 hours, the freshly expressed milk may be added to the previously cooled breast milk under the following conditions:
 - The temperatures must be the same (cold on cold). Freshly expressed milk must therefore be cooled first before being added to milk that is already cool.
- Consult your health visitor, midwife, doctor or chemist for more detailed information about long-term storage.

12. Troubleshooting

Problem	Reason	Remedy
Pump does not work	There is no power	For battery operation: check that the batteries are correctly inserted. For mains operation: check that the mains plug or mains adapter is correctly inserted.
	Lip valve is missing or incorrectly positioned	Check that the lip valve is correctly positioned.
	Pump is being held at an angle	Hold the pump upright. If the pump is held at an angle it will switch off automatically.
Poor pump suction	Low battery power	Check whether there is sufficient battery power. If a light is illuminated beneath the On/Off button the batteries are too weak. Insert new batteries.
	Suction strength is set too low	Turn the suction power regulator towards "+" to attain the optimum suction strength.
Pump suction too powerful	Suction strength is set too high	Turn the suction power regulator towards "-".

13. Technical Data

Medicinal product to 93/42/EEC	Class I
NUK Vented Teat	Complies with EN 14350
Temperature range - Operation - Storage	8 °C to +40 °C 8 °C to +40 °C
Power supply: - Mains operation via adapter Type KK07-045-0800 - Battery operation	Input 100 – 240 V~/50 Hz, output 4.5 VDC, 800 mA, hollow plug: polarity inner +, outer – 3 Type AAA batteries, 1.5 V

14. Disposal

In the European Union, the symbol of a crossed-through wheelie bin means that the product must be disposed of at a separate waste collection point. This applies to this product and all accessories marked with this symbol. Products labelled in this way must not be disposed of with

normal household waste, but should be taken to a collection point for recycling electric and electronic equipment.

Batteries must not be disposed of with household waste. It is every consumer's legal responsibility to ensure that all batteries are disposed of at a specially designated collection point.

15. Warranty

Subject to careful treatment of your appliance and observance of these operating instructions, the manufacturer offers a 24 month warranty from the date you purchase the product.

Appliances sent to us postage paid will be repaired free of charge in the event of faulty materials or workmanship. Any further claims are in accordance with your statutory rights.

No compensation will be given for damage caused by improper use of the device. Wear of the cable or plug is excluded from the warranty. Within the first 24 months after the date of purchase we will rectify any recognised faults without any charge for incidental costs incurred.

Under all circumstances, the device must be returned accompanied by this completed warranty certificate or, failing this, the purchase receipt or other proof of the purchase date must be supplied.

If no proof of purchase date is provided, you will be invoiced for the costs incurred without further inquiry, and the item will be returned to you with a request of payment on delivery.

Please complete the warranty form immediately after purchase and keep this, or the till receipt, as proof of purchase in a safe place. Lost warranty certificates will not be replaced.

Please keep the dealer's address, Art. No. and LOT No. in case of complaints.

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EMC-Guidance

Warnings:

- Medical electrical equipment needs special precautions regarding EMC. The following EMC guidelines must be observed during installing and put into service of the device.

- Portable and mobile RF communications equipment can affect medical electrical equipment.

Guidance and manufacturer's declaration - electromagnetic emissions

The electrical breast pump NUK e-Motion is intended for use in the electromagnetic environment specified below. The customer or the user of the electrical breast pump NUK e-Motion should assure that it is used in such an environment.

Emission test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 14	complies	The electrical breast pump NUK e-Motion is not suitable for interconnection with other equipment.
Harmonic emissions IEC 61000-3-2	Class A	<ul style="list-style-type: none">The electrical breast pump NUK e-Motion is suitable for use in all establishments, including domestic establishments and those directly connected to the public-low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations / flicker emissions IEC 61000-3-3	complies	

**Guidance and manufacturer's declaration - electromagnetic emissions**

The electrical breast pump NUK e-Motion is intended for use in the electromagnetic environment specified below. The costumer or the user of the electrical breast pump NUK e-Motion should assure that it is used in such an environment.

Immunity test	IEC 60601-1 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	+/- 6kV contact +/- 8 kV air	+/- 6kV contact +/- 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic materials, the relative humidity should be at least 30%.
Electrical fast transient/ burst IEC 61000-4-4	+/- 2 kV for power supply lines +/- 1 kV for input/ output lines	+/- 2 kV for power supply lines +/- 1 kV for input/ output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	+/- 1 kV differential mode +/- 2 kV common mode	+/- 1 kV differential mode +/- 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% for 5s <5% for 0,5 cycles 40% for 5 cycles 70% for 25 cycles	<5% for 5s <5% for 0,5 cycles 40% for 5 cycles 70% for 25 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the electrical breast pump NUK e-Motion requires continued operation during power mains interruption, it is recommended that the electrical breast pump NUK e-Motion be powered from an uninterrupted power supply or battery.
Power frequency (50/60Hz) magnetic fields IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of the typical location in a typical commercial or hospital environment.

**Guidance and manufacturer's declaration - electromagnetic emissions**

The electrical breast pump NUK e-Motion is intended for use in the electromagnetic environment specified below. The costumer or the user of the electrical breast pump NUK e-Motion should assure that it is used in such an environment.

Immunity test	IEC 60601-1 test level	Compliance level	Electromagnetic environment - guidance
conducted RF IEC 61000-4-6	3 Veff 150 kHz to 80 MHz	3 V	Portable and mobile RF communications equipment should be used no closer to any part of the Public Access Defibrillator (COR 043), including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended Separation distance: $d = 1,17 * \text{SQRT}(P/W)$
Radiated IEC 61000-4-3	3 V/m 80 MHz to 2,5 GHz	3 V/m	$d = 4 * \text{SQRT}(P/W)$ for 80 MHz to 800 MHz $d = 7,67 * \text{SQRT}(P/W)$ for 800 MHz to 2,5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strength from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the symbol "nonionising radiation".
NOTE: At 80 MHz and 800 MHz the separation distance for the higher frequency range applies.			
NOTE : This guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
Field strength from fixed transmitters, such as base stations for radio (Cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy.			
To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the electrical breast pump NUK e-Motion is used exceeds the applicable RF compliance level above, additional measures may be necessary, such as reorientation or relocating the electrical breast pump NUK e-Motion.			

**Recommended separation distances between
portable and mobile RF communications equipment and the electrical breast pump
NUK e-Motion**

The electrical breast pump NUK e-Motion is intended for use in an electromagnetic environment in which the radiated RF disturbances are controlled. The customer or the user of the electrical breast pump NUK e-Motion can help prevent electromagnetic interference by maintaining a minimal distance between portable and mobile RF communications equipment (transmitters) and the electrical breast pump NUK e-Motion as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter P in W	Separation distance according to frequency of transmitter in m		
	80 MHz to 800 MHz	800 MHz to 2.5 GHz	150 KHz to 80 MHz
	$d=1,17 * \text{SQRT}(P/W)$	$d=2,33 * \text{SQRT}(P/W)$	$d=1,17 * \text{SQRT}(P/W)$
0,01	0,12	0,23	0,12
0,1	0,37	0,74	0,37
1	1,17	2,33	1,17
10	3,70	7,37	3,70
100	11,70	23,30	11,70

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE: At 80 MHz and 800 MHz the separation distance for the higher frequency range applies.

NOTE : This guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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Please also pay regard to other products from the NUK breastfeeding range, for example the

NUK Breast Milk Container (Set of 3 pcs.)

For practical, hygienic storage of breast milk. The same container can be used for expressing milk and feeding:

- Expressing – fits NUK Sensitive and NUK e-Motion Breast Pump
- Storage – suitable for cooling and freezing, with blank field for entering date and time when milk was expressed
- Preparation – heat up milk easily, preferably with the NUK Baby Food Warmer
- Feeding – fits all NUK FIRST CHOICE teats

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For Baby Kft.
Szölöskert u. 5,
2092 Budapest
☎ +36 23 454 867
info@forbaby.hu

IE - IRELAND

Pemberton International
United Drug House, Magna Business Park
Magna Drive, Citywest Road
Dublin 24

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NUK e-MOTION ELECTRIC BREAST PUMP

- Natural and effective** with its 'DUALogic' 2-phase expressing system
- Ergonomic:** 'Optiform' breast shield for an optimum fit to your breast and rib cage
- Gentle:** Easily adjustable suction strength to match your needs
- Practical:** Few parts for swift assembly and easy cleaning

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NUK is a registered trademark owned by MAPA GmbH