Stilleland

NUK Milchpumpe Comfort

NUK Comfort Breast Pump

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Käufer:

Dear Mother,

For babies, a mother's milk is the most important life source there is. This is why NUK supports you with our NUK breastfeeding products, providing practical and convenient aids to meet every requirement of breastfeeding so that you and your baby can enjoy every precious moment together. This practical breast

pump is designed to help you and your baby whenever natural breast-feeding is not possible.

Please allow 5 minutes of your time to carefully read these instructions before using your NUK Comfort Breast Pump for the first time.

We wish you and your baby every happiness.

I. Important Information Before Using for the First Time

Clean and sterilise all parts except the pump unit **2** (see '7. Cleaning and Sterilising').

2. Description

2.1 Intended purpose

The NUK Comfort Breast Pump is an aid to expressing and collecting breast milk.

Once expressed, breast milk can be stored, frozen, warmed and fed to your baby in the NUK Breast Milk Container. Breast Milk can also be expressed directly into any NUK feeding bottle.

2.2 Function

The NUK Comfort Breast Pump is a manual suction pump and offers the following benefits:

- simple to use and requires no power supply
- adjustable suction strength
- the breast pump comes apart for easy cleaning
- silicone cushion provides a gentle massaging effect and softer feel

- imitates your baby's natural suckling action
- a NUK Breast Milk Container is provided to collect your expressed breast milk
- supplied with a stand for safe, convenient storage

2.3 Contraindications

The NUK Comfort Breast Pump should not be used or only used with special care if you are experiencing any problems with your nipples (for example, inflammation or open cuts). If you are unsure at any time about whether to use the NUK Comfort Breast Pump, please consult your doctor.

3. Information for Safety and Use

For your own safety please note the following points:

3.1 General information

- Please read these operating instructions carefully before use. They are part of the appliance and should be kept available at all times.
- If the breast milk is for a premature baby, you must follow your doctor's instructions.

- Do not expose the appliance and accessories to direct sunlight, as ultraviolet radiation and overheating may cause the plastic materials to become brittle.
- Never immerse the pump unit **2** in water and never take it apart as its functionality cannot otherwise be guaranteed.

- The NUK Comfort Breast Pump and accessories are not toys. Use the NUK Comfort Breast Pump and accessories for their intended purpose only (see '2.1 Intended purpose').
- Only use original NUK accessories, as only this will guarantee the pump's proper functioning.
- Using parts from a different manufacturer may prevent proper functioning.
- Please note that any liability or claim under the warranty expires if accessories are used that are not recommended in these operating instructions, or if any parts other than the original spare parts are used for repairs. This also applies to repairs that are not carried out by qualified persons.
- In the event of any malfunction, the pump must be returned to the manufacturer for repair. Failure to do so will invalidate your warranty.

3.2 Hygiene

- When handling breast milk, hygiene is of particular importance to ensure your baby's healthy development.
- Please refer to **Item 7. Cleaning and Sterilising** to avoid infection or bacterial contamination and to avoid damaging the NUK manual breast pump. **The breast pump must be cleaned thoroughly to ensure full suction power.**
- The breast pump must be cleaned and sterilised before each use.**
- The NUK Comfort Breast Pump and accessories must be **cleaned after each use** in order to guarantee functionality and hygiene.

3.3 NUK Breast Milk Container and NUK Bottles

- If the bottle is damaged on the inside (e.g. by scratches), it should be replaced for hygiene reasons.
- All separate parts that are not currently being used must be stored beyond the reach of

children to avoid the danger of their being swallowed.

- Tea or juices may discolour bottles and accessories.
- Do not microwave closed bottles filled with feed. Heating in this way may cause the contents of the bottle to explode or damage to the bottle material. Uneven heating may result in scalding.
- NUK bottles and teats are exclusively compatible with the relevant system, for example NUK First Choice.

3.4 NUK Vented Teat

- Store hygienically (cleaned and dry) at room temperature and away from direct sunlight.
- For hygiene reasons, sterilise the teat for five minutes in boiling water before using for the first time.
- The teat must be cleaned before each use.
- Store the teat in cold sterilising solutions for no longer than the time recommended by the manufacturer to avoid damage to the material of the teat.
- Do not expose teats to direct sunlight, or heat or leave them in sterilising agents (sterilising solution) for longer than the stated times, as this can damage the teat.
- For hygiene and safety, teats should be replaced every 1 to 2 months.

For your child's safety and health

WARNING!

- Always use this product with adult supervision.
- Never use feeding teats as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Check the teat before each use and pull it in all directions – especially when children are teething. At the first signs of damage or weak points the teat should be discarded.

4. Overview of Working Parts (Figure Ⓛ)

- | | |
|---------------------------|---|
| 1 Suction regulator | 9 Valve |
| 2 Pump unit | 10 Connecting element
(only for wide neck bottle Art. no.
10.252.060, 10.749.037, 10.749.038) |
| 3 Swivel clip | 11 NUK Breast Milk Container |
| 4 Lid with intake fitting | 12 Stand for NUK Breast Milk Container |
| 5 Overflow protection | 13 Stand for NUK Standard Bottle |
| 6 Suction cap | 14 NUK Standard Bottle PC 125 ml |
| 7 Silicone cushion | |
| 8 Valve holder | |

5. Tips for Successful Expressing

Correct expressing needs to be practiced. The NUK Comfort Breast Pump is simple and easy to use but you will need to get used to your new pump.

5.1 Frequency of expressing

How often you need to express depends on the reason for expressing.

If you are separated from your child and/or want to maintain milk production you should observe the natural breastfeeding rhythm. You should therefore express 6 to 8 times daily, including once during the night.

In all other cases, you should express entirely according to your personal wishes and requirements.

5.2 Procedure

Express milk from each breast in turn.

- Change to the other breast when the milk flow decreases or ceases.
- The amount of time needed to express milk can vary greatly from one person to another.

The following changeover intervals have proved effective, but times may vary according to the individual:

- right 5 - 7 minutes; left 5 - 7 minutes;
- right 3 - 5 minutes; left 3 - 5 minutes;
- right 2 - 3 minutes; left 2 - 3 minutes.

5.3 Quantity of milk

Do not be surprised if it takes a little longer at first before the milk really starts to flow. This is not unusual, in fact it is perfectly normal, because in general:

milk production adjusts to daily requirements.

The quantity can therefore also vary considerably.

- Be sure to drink enough fluids to stimulate milk production.
- Do not set the suction pressure too high. Higher suction pressure does not produce a larger amount of milk (see figure ⓘ).

5.4 Milk flow reflex

During natural breast feeding, placing the baby on the breast stimulates the milk flow reflex and the milk begins to flow. A similar effect is brought about by the massaging action of the silicone cushion.

You can further stimulate the milk flow reflex if you:

- find a quiet place where you can be fully relaxed as you express the milk;
- adopt a comfortable and relaxed sitting position;
- place a warm compress (e.g. NUK Gel Breast Cushion) on your breast for a few minutes to help promote milk flow;
- make sure you have eye contact with your baby. If that is not possible, look at a photo of your baby;

- the milk flow reflex triggers milk flow on both breasts simultaneously. You can catch breast milk as

it flows from the breast that is not currently being expressed using the NUK Milk Capturing Dish.

6. Dismantling the Breast Pump (Figures ⑩ to ⑪)

Caution! Always hold the bottle upright when it contains milk or the milk may spill out.

- Remove the silicone cushion 7 from the suction cap 6.
- Remove the pump unit 2 from the suction cap 6 (fig. ⑩):
 - turn the pump unit 2 clockwise as far as it will go;
 - continue to turn the pump unit beyond the point of resistance until you hear the swivel clip 3 disengage from the guide for the suction cap 6;
 - pull the pump unit upwards and remove.
- Push the lid 4 upwards and remove (fig. ⑪). This might require some force at first.
- Remove the overflow protection 5 (fig. ⑫).

5. NUK Bottle 14:

Unscrew the suction cap 6 from the NUK Bottle 14.

NUK Breast Milk Container 11:

Unscrew the suction cap 6 from the connecting element 10 and then unscrew the connecting element 10 from the NUK Breast Milk Container 11.

- Place baby bottle 11 or 14 in the appropriate stand so that it cannot fall over.
- Pull the valve holder 8 out of the suction cup (fig. ⑬).
- Remove the valve 9 from the valve holder 8 (fig. ⑭).
- Finally, clean the individual parts as described in '7. Cleaning and Sterilising'.

7. Cleaning and Sterilising

! Hygiene is of most importance when handling breast milk and all parts that come into contact with it. **You should therefore only use a pump that has been cleaned and sterilised.**

All accessories and parts, apart from the pump unit 2 must be thoroughly cleaned in water, sterilised and dried immediately after each use.

7.1 Cleaning

- Clean all parts **except the pump unit 2**, in a sink or washbasin using commercially available washing up liquid.

Do not clean the pump unit 2, teat or bottle 14 in a dishwasher, as the detergent may damage the material.

Only the NUK Breast Milk Container is suitable for dishwashers, but please note that slight discolouration of the material may occur in rare cases.

- Wipe the pump unit 2 with a damp cloth. This cleaning is sufficient because the pump unit does not come into immediate contact with the breast milk.
- A label is provided on the breast milk container for you to write the date on which the milk was expressed. This label can be wiped with a sponge or cloth and written on again.

7.2 Sterilising

Before sterilising, clean as described in '7.1 Cleaning'.

- Sterilise all parts **except the pump unit 2** in the NUK Steam Steriliser or boil for 5 minutes.

Note: When boiling (sterilising) always check that there is sufficient liquid in the pan in order to avoid damaging the teat and bottles (a boiling time of about 5 minutes is sufficient).

The pump unit **2** must **not** be boiled or sterilised, as it contains components that may be damaged by this process.

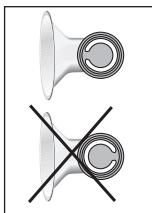
Caution! Do not use a microwave for sterilisation, as the material may be damaged.

Important! The accessories and parts must be allowed to cool fully before using again.

8. Assembly (Figures ④ to ⑩)

1. Place the overflow protection **5** in the upper opening of the suction cap **6** (fig. ④).

Caution: The overflow protection must be inserted so that the connector between the rubber ring and the fastening is pointing to the suction cap.

2. Press the lid **4** firmly onto the overflow protection until it clicks into place all round. It may require a little effort at first to press the lid on completely (fig. ⑤).
3. Position the valve **9** over the valve holder **8** (fig. ⑥).

4. Insert the valve holder into the lower opening in the suction cap **6** (fig. ⑦).

5. For the NUK Breast Milk Container only: Screw the connecting element **10** onto the NUK Breast Milk Container.

6. Screw the suction cap **6** onto the baby bottle (fig. ⑧).

7. Take the pump unit **2** and push the intake fitting of the lid **4** **completely** into the suction connector on the pump unit **2** (fig. ⑨).

8. Push the pump unit **2** firmly onto the suction cap **6** until the swivel clip **3** clicks audibly into place in the suction cap (fig. ⑩).

9. Position the silicone cushion **7** on the suction cap funnel **6** (fig. ⑪).

9. Operation (Figures ⑫ to ⑬)

1. Please ensure your hands and breast are always clean before using the NUK Comfort Breast Pump. To be sure of the necessary hygiene, you should wash your hands thoroughly on each occasion before expressing.
2. Turn the pump unit to the position that is most comfortable for you then gently press onto the pump unit from above (fig. ⑫).
3. Position the suction cap funnel on the breast so that the nipple is in the centre of the silicone cushion **7**. This prevents the nipple becoming sore.

The bottle must be held vertically downwards so that the milk flows into the bottle and not into the pump unit.

4. Press the pump handle and then release it again slowly. At the same time, press the suction cap gently against the breast so that the silicone cushion fits closely and no air is sucked in from the side.

5. Set the suction strength by turning the suction regulator I so that the pumping action is comfortable and pleasant for you (fig. ④).

Please note the information in the section '5. Tips for Successful Expressing'.

6. After expressing, place the breast pump in the stand. The pump must not be allowed to topple over as the milk may run out.

Caution: The piston seal must be oiled regularly with a little baby oil to ensure it slides properly. This prevents the pump becoming stiff (fig. ①).

10. Hygiene Tips

Hygiene is especially important when using the breast pump. This applies to all the individual parts of the pump as well as to your own personal hygiene.

NUK Breast Milk Container

Note: When freezing breast milk, do not fill the container with more than 150 ml/5 fl.oz., as breast milk expands as it freezes and the container may burst.

1. Unscrew the pump unit from the NUK Breast Milk Container.
2. Close the NUK Breast Milk Container firmly with the screw ring and the sealing disc supplied.
3. Use a water-soluble ink pen to write the date and time of expressing on the label provided on the container.

We recommend a daily shower or wash your nipples in clean water. Where possible, avoid using soap on your nipples as this may cause irritation.

11. Storing Milk

Caution! Only the bottle with its lid should be stored in the refrigerator, not the entire NUK Comfort Breast Pump.

11.1 NUK Bottle

If the breast milk is not drunk immediately, you should seal the filled bottle well and store it in the refrigerator. If the breast milk is to be stored for longer than 72 hours it must be frozen immediately in a suitable, hygienically clean container. The NUK Standard Bottle is not suitable for this.

Tip:

Freeze the milk in the NUK Breast Milk Container. Place the container in the refrigerator to defrost, or in warm water if the milk is required quickly.

11.2 NUK Breast Milk Container

The breast milk may be chilled, frozen and defrosted in the breast milk container. The container may also be used for warming and feeding. It is important that you note the information provided below about handling breast milk.

11.3 Tips for handling breast milk

- Breast milk may be stored in a refrigerator (not in the refrigerator door) for up to 72 hours at +4 °C.
- The maximum storage time in the freezer compartment of a refrigerator is 2 weeks and in a freezer it is 6 months at – 18 °C.
- Place the bottle in the refrigerator to defrost, or in warm water if the milk is required quickly.

- To achieve the correct feed temperature of about 37 °C, place the container in a NUK Baby Food Warmer.
- **Caution:** Do not use a microwave to defrost breast milk as this can destroy valuable nutrients, damage the containers material and may create a risk of explosion. Uneven heating may result in scalding! The milk must be shaken well prior to feeding to mix the milk and milk fat again and to distribute the warm breast milk evenly.
- Defrosted, warmed or unused milk must not be refrozen and must be thrown away.
- Consult your nursing advisor, midwife, doctor or chemist for more detailed information about long-term storage.

12. Disposal

If at some stage the appliance is no longer fit for use, the parts must be disposed of in accordance with local regulations.

13. Troubleshooting

Problem	Reason	Remedy
Pump has no suction or too little suction.	<ul style="list-style-type: none">• Lid 4 is not fitted correctly or the overflow protection 5 is wrongly inserted.• Valve 9 is fitted wrongly on the valve holder 8.• Suction regulator dial 1 is not fitted correctly.• Pump unit is not fitted correctly: intake fitting on lid 4 is not fitted correctly in the suction connector on the pump unit 2.	<ul style="list-style-type: none">• Check that the overflow protection is correctly positioned;• Ensure that the lid has been pressed on fully (see Section 8.).• Check the correct fit (see Section 8.).• Pull the dial off, rotate slightly, and press back on.• Remove pump unit 2 and replace. Check that the intake fitting on the lid 4 is fully inserted into the suction connector on the pump unit 2 (see Section 8.).
Pump lever feels stiff.	<ul style="list-style-type: none">• Too much friction on the piston seal.• Overflow protection 5 is wrongly oriented.	<ul style="list-style-type: none">• Remove pump unit 2. Press hand lever as far as possible. Apply a little baby oil to the piston seal.• The connecting piece must point towards the suction funnel (see Section 8.).
Suction regulator feels stiff.	<ul style="list-style-type: none">• Too much friction on the suction regulator.	<ul style="list-style-type: none">• Pull the dial off. Apply a drop of baby oil to the suction regulator. Press the dial back on.

14. Technical Data

Medical product in accordance with 93/42/EEC	Class I
Temperature range	
– Operation	6 °C to +40 °C
– Storage	0 °C to +40 °C

15. Warranty

Subject to careful treatment of your appliance and observance of these operating instructions, the manufacturer offers a 24 month warranty from the date you purchase the appliance.

Appliances sent to us postage paid will be repaired free of charge in the event of faulty materials or workmanship. Any further claims are in accordance with your statutory rights.

Any further claims are in accordance with your statutory rights.

No compensation will be given for damage caused by improper use of the appliance. Within the first 24 months after the date of purchase, we will

rectify recognised defects without any charge for incidental costs incurred.

Under all circumstances, the appliance must be returned accompanied by this completed warranty certificate or by another proof of purchase date. If no proof of purchase date is provided, you will be invoiced for the costs incurred without further inquiry and the appliance returned to you with a request of payment on delivery.

Please complete the warranty certificate immediately after purchase and keep it safe.

Lost warranty certificates will not be replaced.
Please keep distributor's address, Art. No. and Lot No. in case of complaints.

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NUK is a registered trademark
owned by MAPA GmbH