Stillglück

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NUK®
SEIT 1956 EST.

Dear Mother,

For babies, a mother's milk is the most important lifesource there is. That is why NUK supports you with the NUK breastfeeding programme, providing practical and convenient aids to meet every requirement of breastfeeding, so that you and your baby can enjoy every precious moment together.

This easy-to-handle breast pump is designed to help you and your baby whenever natural breast-feeding is not possible.

Please allow 5 minutes of your time to read these instructions for use carefully before using your NUK Sensitive Breast Pump for the first time.

We wish you and your baby every happiness.

1. Important Information Before Using for the First Time

Clean and sterilise all parts except the pump unit **2** before using the NUK Sensitive Breast Pump for the first time (see "7. Cleaning and Sterilising").

2. Description

2.1 Intended purpose

The NUK Sensitive Breast Pump is an aid to expressing and collecting breast milk:

- in the workplace or when baby is away, at home or when travelling
- to maintain milk production during a break in breast feeding
- to relieve engorgement.

Once expressed, breast milk can be collected, stored, frozen, warmed and fed to your baby in the NUK Breast Milk Container.

2.2 Function

The NUK Sensitive Breast Pump is a manual suction pump and offers the following benefits:

- simple to use and needs no power supply
- individually adjustable suction strength
- breast pump comes apart for easy cleaning
- silicone cushion for gentle massaging effect and softer feel
- imitates your baby's natural suckling action
- collects expressed breast milk
- supplied with stand for safe, convenient storage

2.3 Contraindications

In the event of certain nipple problems (inflammation, open cuts) the NUK Sensitive Breast Pump should not be used or only used with special care. If you are unsure at any time about whether to use the NUK Sensitive Breast Pump please consult your doctor.

3. Information for Safety and Use

For your own safety please note the following points:

3.1 General Information

- Please read these operating instructions carefully before use. They are part of the device and must be kept available at all times.
- Use the NUK Sensitive Breast Pump and accessories for their intended purpose only (see "2.1 Intended purpose").
- Never immerse the pump unit **2** in water and never take it apart, as its functionality cannot otherwise be guaranteed.
- Keep small parts that may be swallowed out of the reach of children: there is a danger of choking.
- If ointments or creams are used, please check in the manufacturer's information whether the ointment or cream needs to be removed before nursing or expressing milk.
- Only use original NUK accessories, as only this will guarantee the pump's proper functioning.
- If accessories from other manufacturers are used, the pump may not function properly.
- Please note that any liability or claim under the warranty expires if accessories are used that are not recommended in these operating instructions, or if any parts other than original spare parts are used for repairs. This also applies to repairs that are not carried out by qualified persons.
- In the event of any malfunction, the pump must be returned to the manufacturer for repair. Failure to do so will invalidate your warranty.
- If the breast milk is intended for a premature baby, it is essential that you follow your doctor's advice.

3.2 Hygiene

- When handling breast milk, hygiene is of particular importance to ensure your baby's healthy development.
- Please refer to Section "7. Cleaning and Sterilising" on page 16 to prevent infection or bacterial contamination and to prevent the device from becoming damaged.
- **Only if the breast pump is cleaned thoroughly will it attain its full suction power.**

3.3 NUK Breast Milk Container

- If the bottle is damaged on the inside (e.g. by scratches), it should be replaced for hygiene reasons.
- All unused individual parts should always be kept out of the reach of children as there may be a danger of choking.
- Tea or juices may discolour bottles and accessories.
- Give only the quantities of food and drink recommended by the manufacturer and in the recommended consistency.
- Sterilisation in the microwave is not recommended as this can damage the materials.
- Feeding bottles that are sealed and contain food should not be put in the microwave as there may be a risk of explosion or material damage. Heating in this way could cause the contents of the bottle to explode. Uneven heating may result in scalding.
- NUK bottles and teats are only compatible with the relevant system, NUK First Choice.

3.4 NUK Vented Teat

- Clean and sterilise the teat thoroughly before using for the first time and before each further use.
- The NUK Steam Steriliser is recommended for sterilising.
- When boiling, check that there is sufficient liquid in the steriliser in order to avoid damage (5 minutes boiling time is sufficient).
- The teat must not be cleaned in a dishwasher or sterilised in a microwave as it could be damaged by the dishwasher detergent or deformed.

For your child's health and safety:

CAUTION!

- Store in a hygienic condition and away from light sources.
- Always check food temperature before feeding.
- This product must only be used under adult supervision. A teat should never be used as a

soother. Continuous and prolonged sucking of fluids may cause tooth decay.

- Do not dip the teat in sweetened liquids or medicine.
- **Check the teat before each use by pulling it in all directions – especially when children are teething. Discard the teat at the first sign of tearing or biting damage as there is a danger that small parts may become detached and swallowed.**
- Do not leave teats in cold sterilising solution beyond the period specified by the manufacturer, as this may cause damage to the material.
- For hygiene and safety reasons, teats should be replaced every 1-2 months.
- Use teats only with NUK FIRST CHOICE bottles. The NUK Vented Teat complies with EN 14350.

4. Overview of Working Parts (Figure ①)

- | | |
|------------------------------|--|
| 1 Suction strength regulator | 6 Suction cap |
| 2 Pump unit | 7 Silicone cushion |
| 3 Coupler | 8 Lip valve |
| 4 Lid with intake fitting | 9 NUK Breast Milk Container |
| 5 Overflow protection | 10 Stand for NUK Breast Milk Container |

5. Tips for Successful Expressing

Correct expressing needs practice. You should first become accustomed to your new pump.

5.1 Frequency of expressing

The frequency of expressing depends on individual requirements.

If you will be apart from your child and/or want to sustain milk production, the natural breastfeeding rhythm should be maintained. You should therefore express 6 to 8 times daily, including once during the night.

In all other cases you should express entirely to suit your own individual needs.

5.2 Procedure

Express milk from each breast in turn.

- Alternate between breasts when the milk flow slows down or ceases.
- The amount of time needed to express milk will vary considerably from one person to another.

The following intervals have proved effective, but times may vary according to the individual:

- right 5 - 7 minutes; left 5 - 7 minutes;
right 3 - 5 minutes; left 3 - 5 minutes;
right 2 - 3 minutes; left 2 - 3 minutes.

5.3 Milk quantity

Do not be surprised if it takes a little longer at first before the milk really starts to flow. This is not unusual, in fact it is perfectly normal, because in general:

milk production adjusts to daily requirements.

This means that the quantity of milk produced may vary considerably.

- Be sure to drink enough fluids to stimulate milk production.
- Do not set the suction pressure too high. Higher suction pressure does not produce a larger amount of milk (see figure ⑧).

5.4 Milk flow reflex

During natural breast feeding, placing the baby on the breast stimulates the milk flow reflex and the milk begins to flow. A similar effect is brought about by the massaging action of the silicone cushion.

You can further stimulate the milk flow reflex if you:

- find a quiet place where you can be fully relaxed as you express the milk.
- adopt a comfortable and relaxed sitting position.
- place warm compresses (e.g. NUK Gel Breast Cushion) on your breast for a few minutes to relax the tissue before expressing.
- make sure you have eye contact with your baby. If that is not possible, look at a photo of your baby.
- The milk flow reflex releases the flow of milk to both breasts simultaneously. Any surplus milk leaking from the breast that is not currently being expressed can be collected using the NUK Milk Capturing Dish.

6. Dismantling the Breast Pump (Figures ① to ⑧)

Caution! Always hold the bottle upright when it contains milk or the milk may spill out.

1. Remove the silicone cushion **7** from the suction cap **6** (fig. ①).
2. Remove the pump unit **2** from the suction cap **6** (fig. ②):
– lift the pump unit **2** upwards;
– pull the pump unit upwards and remove.
3. Push the lid **4** upwards and remove (fig. ③). This might require some force at first.
4. Remove the overflow protection **5** (fig. ④).
5. Unscrew the suction cap **6** from the NUK Breast Milk Container **9**.
6. Place the NUK Breast Milk Container in the stand so that it cannot fall over.

7. Pull the lip valve **8** out of the suction cap (fig. ⑤).
8. Finally, clean the individual parts as described in “7. Cleaning and Sterilising”.

Stillglück

7. Cleaning and Sterilising

Hygiene is of priority when handling breast milk and all parts that come into contact with it. **The pump should therefore only be used in a clean**

and sterile condition.

All accessories and parts, apart from the pump unit 2, must be thoroughly cleaned in water, sterilised and quickly dried immediately after each use.

7.1 Cleaning

1. Clean all parts, **except the pump unit 2**, in a **sink or washbasin** using commercially available washing up liquid.

Only the NUK Breast Milk Container is suitable for dishwashers, but please note that slight discoloration of the material may occur in rare cases.

2. Wipe the pump unit 2 with a damp cloth. This cleaning is sufficient because the pump unit does not come into immediate contact with the milk.

3. A label is provided on the breast milk container for you to write the date on which the milk was expressed. This label can be wiped with a sponge or cloth and written on again.

7.2 Sterilising

Before sterilising, clean as described in “7.1 Cleaning”.

1. Sterilise all parts **except the pump unit 2**:
 - in the NUK Steam Steriliser or
 - boil for 5 minutes.

Note: When boiling (sterilising) always check that there is sufficient liquid in the pan in order to avoid damaging the teat and bottles (a boiling time of about 5 minutes is sufficient).

The pump unit 2 must **not** be boiled or sterilised, as it contains components that could be damaged by this process.

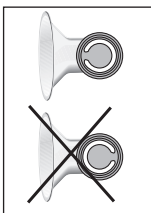
Caution! Do not use a microwave for sterilisation, as otherwise the material may be damaged.

Important! The accessories and parts must be allowed to cool fully before using again.

8. Assembly (Figures a to f)

1. Place the overflow protection 5 in the upper opening of the suction cap 6 (fig. a).

Caution: The overflow protection must be inserted so that the connector between the rubber ring and the fastening is pointing to the suction cap.

2. Press the lid 4 firmly onto the overflow protection until it clicks into place all the way round. It may require a little effort at first to press the lid on completely (fig. a).

3. Put the lip valve 8 onto the lower opening in the suction cap 6 (fig. b).
4. Screw the suction cap 6 onto the breast milk container (fig. c).

5. Take the pump unit 2 and push the intake fitting of the lid 4 **completely** into the suction connector on the pump unit 2 (fig. d).

6. Push the pump unit 2 firmly onto the suction cap 6 until the coupler 3 clicks audibly into place in the suction cap (fig. e).
7. Position the silicone cushion 7 on the suction cap funnel 6 (fig. f).

9. Operation (Figures g to h)

1. Please ensure your hands and breast are always clean. To be sure of the necessary hygiene, you should wash your hands thoroughly on each occasion before expressing.
2. Position the suction cap funnel on the breast so that the nipple is in the centre of the silicone cushion 7. This prevents the nipple becoming sore.
The bottle must be held vertically downwards so that the milk flows into the bottle and not into the pump unit.
3. Press the pump handle and then release it again slowly. At the same time, press the suction cap gently against the breast so that the silicone cushion fits closely and no air is sucked in from the side.
4. Set the suction strength by turning the suction regulator I so that the pumping action is comfortable and pleasant for you (fig. g).

Please note the information in the section "5. Tips for Successful Expressing".

5. After expressing, place the breast pump in the stand. The pump must not be allowed to topple over as milk might run out.

Caution: The piston seal must be oiled regularly with a little baby oil to ensure it slides properly. This prevents the pump becoming stiff (fig. h).

NUK Breast Milk Container

1. Unscrew the pump unit from the NUK Breast Milk Container.
2. Close the NUK Breast Milk Container firmly with the screw ring and sealing disc supplied.
3. Use a freezer pen to write the date and time of expressing on the label provided on the container.

10. Hygiene Tips

Hygiene is particularly important when using the breast pump. This applies to all the individual pump components as well as to your own personal hygiene.

We recommend a daily shower or wash your nipples in clean water. Where possible, avoid using soap on your nipples, as this may cause irritation.

11. Storing Milk

If the breast milk is not to be used immediately, you should seal the filled bottle well and store it in the refrigerator.

Caution! Only the bottle with its lid should be stored in the refrigerator, not the entire NUK Sensitive Breast Pump.

11.1 NUK Breast Milk Container

The breast milk may be chilled, frozen and thawed out again in the breast milk container. The container may also be used for re-heating and feeding. Please always observe the information below on handling breast milk.

Note: When freezing breast milk, do not fill the container with more than 150 ml (5 fl.oz), as breast milk expands as it freezes and the container could burst.

11.2 Tips for Handling Breast Milk

- Breast milk may be stored in a refrigerator (but not in the refrigerator door) for up to 72 hours at +4 °C.
- The maximum storage time in the freezer compartment of the refrigerator is 2 weeks, and in a freezer it is 6 months at – 18 °C.
- If transporting chilled breast milk make sure that it stays chilled at all times otherwise the milk could turn sour and be harmful to your baby. Use a cooler bag, for example, with ice packs.
- Thawed milk may be stored **unopened** at 4 °C for up to 24 hours.
- Once the container has been opened, the maximum storage time is 12 hours at 4 °C.
- The milk may be thawed out in the bottle either in the refrigerator, or, if required immediately, by standing in warm water.
- To attain the correct feed temperature of approx. 37 °C, place the container in a NUK Baby Food Warmer.
- **Caution:** Breast milk must not be thawed out in a microwave as this will destroy valuable nutrients as well as causing possible damage to the container material, which may cause the risk of explosion. Uneven heating may result in scalding!
- The milk should therefore be shaken gently before feeding to ensure that milk and milk fats are well mixed and that the warm breast milk is evenly distributed.
- Defrosted, re-heated or unused milk must be disposed of and not refrozen.
- If you express breast milk more than once within a period of 24 hours, the freshly expressed milk may be added to the previously cooled breast milk under the following conditions:
 - The temperatures must be the same (cold on cold). Freshly expressed milk must therefore be cooled first before being added to milk that is already cool.
- Consult your health visitor, midwife, doctor or chemist for more detailed information about long-term storage.

12. Disposal

If at some stage the appliance is no longer fit for use, the parts can be disposed of along with normal household waste.

13. Troubleshooting

Problem	Reason	Remedy
Pump has no suction or too little suction	• Lid 4 is not fitted correctly	• Ensure that the lid has been fully pressed on (see Section 8.).
	• Overflow protection 5 is wrongly inserted	• Check that the overflow protection is correctly positioned and aligned.
	• Suction regulator dial 1 is not fitted correctly	• Pull the dial off, rotate slightly, and press back on.
	• Pump unit is not fitted correctly: intake fitting on lid 4 is not fitted correctly in the suction connector on the pump unit 2	• Remove pump unit and replace. Check that the intake fitting on the lid 4 is fully inserted into the suction connector on the pump unit 2 (see Section 8.).
	• Lip valve is incorrectly positioned	• Position the lip valve correctly (see Section 8.).
	• Pump is being held at an angle	• Hold the pump as upright as possible.
Pump lever feels stiff	• Too much friction on the piston seal	• Remove pump unit 2 . Press hand lever as far as possible. Apply a little baby oil to the piston seal.
	• Overflow protection 5 is wrongly oriented	• The connecting piece must point towards the suction funnel (see Section 8.).
Suction regulator feels stiff	• Too much friction on the suction regulator	• Pull the dial off. Apply a drop of baby oil to the suction regulator. Press the dial back on.

14. Technical Data

Medical product in accordance with 93/42/EEC	Class I
Temperature range	
– Operation	6 °C to +40 °C
– Storage	0 °C to +40 °C

Stillglück

15. Warranty

Subject to careful treatment of your appliance and observance of these operating instructions, the manufacturer offers a 24 month warranty from the date you purchase the appliance.

Appliances sent to us postage paid will be repaired free of charge in the event of faulty materials or workmanship.

Any further claims are in accordance with your statutory rights.

No compensation will be given for damage caused by improper use of the appliance. Within the first 24 months after the date of purchase we will rectify any recognised faults without any charge for incidental costs incurred.

The appliance must under all circumstances be returned accompanied by this completed warranty certificate or by another proof of the purchase date.

If no proof of purchase date is provided, you will be invoiced for the costs incurred without further inquiry, and the item will be returned to you with a request of payment on delivery.

Please complete the warranty certificate immediately after purchase and keep it safe. Lost warranty certificates will not be replaced.

Please keep the dealer's address, Art. No. and LOT No. in case of complaints.

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IE - IRELAND

Pemberton International
United Drug House, Magna Business Park
Magna Drive, Citywest Road
Dublin 24

IT - ITALY

MAPA GmbH
Industriestraße 21-25
D-27404 Zeven
Germania
e-mail: nuk-italia@mapa.de

LT - LITHUANIA

UAB Sabinos sandeliai
Europos pr. 36
46369 Kaunas
☎ +370 37 39 11 38

LV - LATVIA

Amajia Ltd.,
J.Asara Street 5
1009 Riga
☎ +371 70 40 782

MK - MACEDONIA

Euroalba Skopje d.o.o.
Bul. Partizanski Odredi 64 g 3/6
1000 Skopje
☎ +389 2 30 65 717

NO - NORWAY

Simba-Dickie Nordic AS
Postboks 34
3161 Stokke
☎ (0047) 33 30 53 00
Fax: (0047) 33 30 53 10
e-mail: mail@simbatoy.no

Babycare AS
C/O DHL Terminal
Stemmanne 10
N-4697 KRISTIANSAND S

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SENSITIVE BREAST PUMP

- Easily adjustable – Alter the suction strength to suit your needs
- Super-soft, contoured silicone cushion – moulds to the shape of your breast and gently massages
- Ergonomic handle – for easy expressing with one hand

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27404 Zeven
Germany
www.nuk.de

NUK is a registered trademark
owned by MAPA GmbH

Matr.-Nr. 10.520.120
REV 101

NUK
SEIT 1956 EST.